




# ImmunoCard **STAT! NOROVIRUS**


Rapid test for the detection of Norovirus in stool specimens

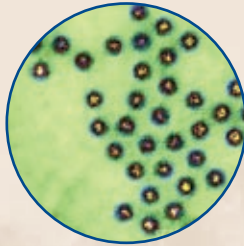


## Be ahead of the outbreaks with routine testing

- Rapid detection of the most common cause of gastroenteritis in people of all ages
- Prompt diagnosis of potential outbreaks
- Results can be useful in outbreak containment and infection control
- One step test provides results in 15 minutes
- Simultaneous detection of Genogroups I and II
- Room temperature storage


## Test Procedure

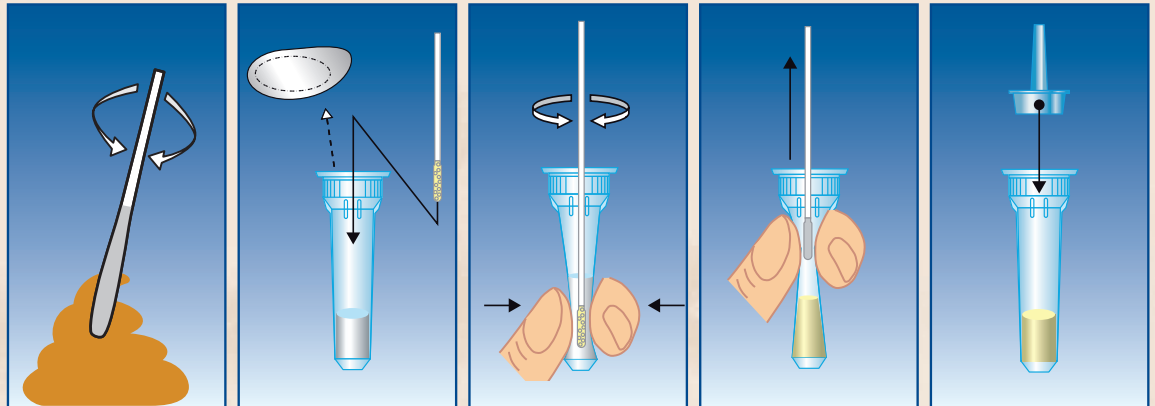

1. Collect the stool specimen with the swab supplied with the kit.
2. Remove the aluminium seal from the Sample Diluent Tube. Immediately place the swab into the Sample Diluent Tube.
3. Vigorously mix the solution by rotating the swab against the side of the Sample Diluent Tube.
4. Release as much liquid as possible from the swab by squeezing the sides of the tube as the swab is withdrawn. Discard the swab.
5. Firmly attach the Dropper Cap with Filter to the Sample Diluent Tube.

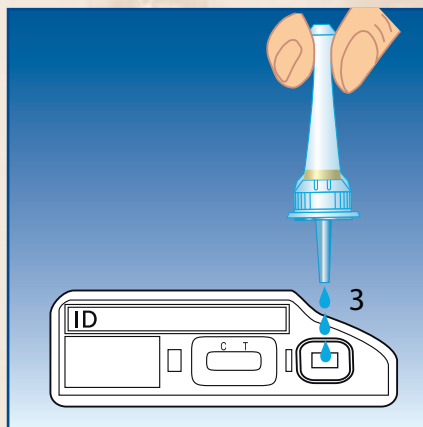

6. Turn the tube upside down and dispense 3 drops of diluted specimen to Sample port of the Test Card. Incubate 15 minutes at 15-30°C.

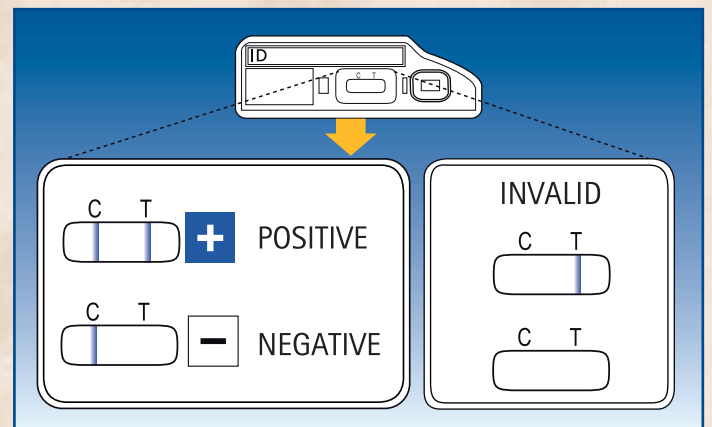

7. Read Control (C) and Test (T) zones for the presence or absence of a blue line.

## Ordering Information

Products	Test Size	Catalog #
<i>ImmunoCard STAT!® NOROVIRUS</i>	20 tests	757020
<i>ImmunoCard STAT!® NOROVIRUS Positive Control</i>	1 test	757001
<i>Rapid Strip ROTA/ADENO</i>	20 tests	751120
<i>ImmunoCard STAT!® ROTAVIRUS</i>	30 tests	750030
<i>ImmunoCard STAT!® ADENOVIRUS</i>	30 tests	751030
<i>Premier™ Rotaclone®</i>	48 Microwells	696004
<i>Premier™ Adenoclone®</i>	48 Microwells	696007
<i>Premier™ Adenoclone® - Type40/41</i>	48 Microwells	696006



**Meridian Bioscience Europe Srl**  
Via dell' Industria, 7  
I-20020 Villa Cortese, Milano - Italy  
Tel: +39 (0331) 43 36 36  
Fax: +39 (0331) 43 36 16  
E-mail: info@mdeur.com

**Meridian Bioscience Europe sa/nv**  
Rue de l'Industrie, 7  
B-1400 Nivelles - Belgium  
Tel: +32 (67) 89 59 56  
Fax: +32 (67) 64 41 92  
E-mail: info@mdeur.be

**Meridian Bioscience Europe France**  
Le Quadra Promenade des Anglais, 455  
F-06299 Nice Cedex 3 - France  
Tel: +33 (4) 93 18 72 10  
Fax: +33 (4) 93 18 72 11  
E-mail: info@meridianbioscience.fr

**Meridian Bioscience Europe bv**  
Halderheiweg, 6  
NL-5282 SN Boxtel  
The Netherlands  
Tel: +31 (411) 62 11 66  
Fax: +31 (411) 62 48 41  
E-mail: meridian.info@planet.nl

**Meridian Bioscience, Inc.**  
Corporate Office  
3471 River Hills Drive  
Cincinnati, Ohio 45244, USA  
Tel: +1 (513) 271 37 00  
Fax: +1 (513) 271 37 62  
E-mail: mbi@meridianbioscience.com

## DISTRIBUTED BY

[www.mdeur.com](http://www.mdeur.com)  
[www.meridianbioscience.com](http://www.meridianbioscience.com)